

Imagine

whipped cream...



...then realize that it is **JET BLUE® Bite**
Bite Registration Material

- Stays exactly where you want it
- No resistance to closure, no rebound!
- Rock hard for accuracy, yet flexible for easy trimming
- Pick your ideal speed
 - **Fast** - 30 second working time
40 second oral setting time
 - **Superfast** - 20 second working time
30 second oral setting time



coltène 
whaledent®

Exceptional Handling with firm, accurate, fast set

JET BLUE® Bite fast / superfast bite registration material flows with a smooth whipped-cream consistency exactly where you want it in a predictable quantity and ideal ribbon pattern. It offers a fast oral setting

time and sets rock hard for greater accuracy but is still flexible enough for easy trimming or cutting. The superior handling characteristics are the key for a perfect occlusal fit on the first try.

Direct application of JET BLUE® Bite fast



Application of Jet Blue® Bite fast



Direct application with the unique Spreader Tip



Ideal working-time



Bite registration




Final bite registration



Easy cutting



Easy trimming

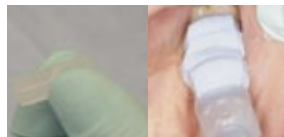

Important and helpful accessories: Unique spreader tip for predictable and consistent dispensing ribbon width and depth

Try JET BLUE® Bite in the 25ml microSystem™. 38% less waste in the mixing tips!



JET BLUE® Bite	Working time	Oral setting time	Endhardness	Dimensional stability in %
JET BLUE® Bite fast	max. 30 s	min. 40 s	89 Shore A	-0,08%
JET BLUE® Bite superfast	max. 20 s	min. 30 s	89 Shore A	-0,12%

Internal measurement

JET BLUE® Bite fast microSystem™ Single Pack	6492
Cartridges	2 x 25 ml
MT/CLEAR/CLEAR/WHITE/70/S	10 pcs.
Spreader Tips	10 pcs.
JET BLUE® Bite fast microSystem™ Refill	6496
Cartridges	8 x 25 ml
MT/CLEAR/CLEAR/WHITE/70/S	20 pcs.
Spreader Tips	20 pcs.
JET BLUE® Bite fast 50 ml Single Pack	6491
Cartridge	1 x 50 ml
MT/PINK/CLEAR/WHITE/70/S	6 pcs.
Spreader Tips	6 pcs.
JET BLUE® Bite fast 50 ml Refill	6495
Cartridges	4 x 50 ml
MT/PINK/CLEAR/WHITE/70/S	12 pcs.
Spreader Tips	12 pcs.

JET BLUE® Bite superfast microSystem™ Single Pack	6494
Cartridges	2 x 25 ml
MT/CLEAR/CLEAR/WHITE/70/S	10 pcs.
Spreader Tips	10 pcs.
JET BLUE® Bite superfast microSystem™ Refill	6498
Cartridges	8 x 25 ml
MT/CLEAR/CLEAR/WHITE/70/S	20 pcs.
Spreader Tips	20 pcs.
JET BLUE® Bite superfast 50 ml Single Pack	6493
Cartridge	1 x 50 ml
MT/PINK/CLEAR/WHITE/70/S	6 pcs.
Spreader Tips	6 pcs.
JET BLUE® Bite superfast 50 ml Refill	6497
Cartridges	4 x 50 ml
MT/PINK/CLEAR/WHITE/70/S	12 pcs.
Spreader Tips	12 pcs.

JET BLUE® Bite Accessories		
microSystem™		
MT/CLEAR/CLEAR/WHITE/70/S	40 pcs.	6214
MT/CLEAR/CLEAR/WHITE/70/S	100 pcs.	6215
Spreader Tips	100 pcs.	6223
microSystem™ Dispenser MK II	1 pc.	6270

JET BLUE® Bite Accessories		
50 ml System		
MT/PINK/CLEAR/WHITE/84/S	40 pcs.	6551
Spreader Tips	100 pcs.	6223
Dispenser 50/75	1 pc.	6900

* MT = Mixing Tip
S = straight

www.coltenewhaledent.com
© 2006 Coltène/Whaledent

Coltène/Whaledent AG
Feldwiesenstrasse 20
9450 Altstätten/Switzerland
Tel. +41 (0)71 757 53 00
Fax +41 (0)71 757 53 01
office@coltenewhaledent.ch

Coltène/Whaledent Ltd.
The President Suite-A
Kendal House, Burgess Hill
Victoria Way
West Sussex, RH15 9NF/U.K.
Tel. +44 (0)1444 235486
Fax +44 (0)1444 870640
hq@coltenewhaledent.co.uk

Coltène/Whaledent Inc.
235 Ascot Parkway
Cuyahoga Falls, Ohio 44223/USA
Tel. +1 330 916 8800
Fax +1 330 916 7077
productinfo@coltenewhaledent.com

coltène
whaledent