


Cleaning and disinfection of surfaces

as at: 8 July 2015

Large dry wipes in the MaxiWipes-Box for the cleaning and disinfection of surfaces of medical devices in combination with MinutenSpray-classic or PlastiSept.

- *nonwoven tearproof wipes of high quality*
- *optimal application of active solution*
- *effective cleaning action by means of micro pore technology*
- *large / extra large wipes:*
 - *MaxiWipes: 17x26 cm,*
 - *MaxiWipes-L: 20x40 cm**(ideal for large surfaces)*
- *suitable for all purposes (with MinutenSpray-classic or PlastiSept)*

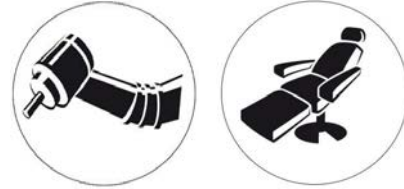

MaxiWipes Box

MaxiWipes-L Box

Cardboard box with 4 refill reels (of MaxiWipes or MaxiWipes-L)

Application



*To soak the dry wipes use either PlastiSept** (alcohol-free) or MinutenSpray-classic** (alcoholic).

1. Remove the lid of the MaxiWipes Box. (To refill put the reel into the box).
2. Soak the dry reel with the requested disinfection solution.

MaxiWipes/reel:	1,5 ltr. PlastiSept or 2,5 ltr. MinutenSpray-classic
MaxiWipes-L/reel:	1,5 ltr. PlastiSept or 2,5 ltr. MinutenSpray-classic
3. Pull the first wipe through the tear-off slot of the lid, close the box and allow the wipes to get soaked for 30 minutes. Then the wipes are soaked sufficiently and ready for use.

After being soaked by MinutenSpray-classic or PlastiSept the MaxiWipes are usable for at least 28 days – provided that the dispenser lid of the box is always closed tightly.

**TGA-registered or listed

Cleaning and disinfection of surfaces

as at: 8 July 2015

Please note:

The label of the MaxiWipes Box has to be completed correctly!
Please mark the checkbox referring to the used soaking fluid with a cross.
Additionally, we recommend fixing the attached small label indicating the details
as to shelf life and lot number of the used soaking fluid!

Usage only by qualified personnel.

Soaking liquids:

PlastiSept**

ARTG No. 164389



MinutenSpray-classic**

ARTG No. 161366



Composition

< 5 % cationian tensides, deion. water, fragrance, emulsifying agent. Disinfecting substances: quaternary alkylammonium chlorides

Composition

100 g contain: 45 g ethanol, 15 g isopropanol, 0,1 g chlorhexidine-digluconate, 0,08 g trialkylethoxyammoniumpropionate, 0,05 g alkylamine derivates

Efficacy

- *bactericidal against Staphylococcus aureus (incl. MRSA), Pseudomonas aeruginosa, Escherichia coli, Proteus vulgaris, Salmonella choleraesuis.*
- *fungicidal against Trichophyton mentagrophytes.*
- *virucidal against Human Influenza Virus, Herpes simplex virus type 1 (HSV).*

Efficacy

bactericidal against Staphylococcus aureus (incl. MRSA), Pseudomonas aeruginosa, Salmonella choleraesuis.

Times of effect (undiluted)

bactericidal	in 10 minutes
fungicidal	in 10 minutes
virucidal	in 10 minutes

Times of effect (undiluted)

bactericidal	in 10 minutes
--------------	---------------

Clinical data

- Siliker, Melbourne, TGA Test for Hospital grade Disinfectant under `dirty` conditons (Option B), 2009-02
- Microbiotest, Sterling Virginia, Virucidal Efficacy Test – Human Influenza Virus, 2005-02; Virucidal Efficacy Test Herpes Simplex Virus type 1, 2005-01; AOAC, Germicidal Spray Test using Fungi, 2004-07; AOAC Germicidal Spray Test, 2004-04

Please feel free to request these clinical data.

Clinical data

Siliker, Melbourne, AOAC method 961.02 (Germicidal Spray Products as Disinfectant) in accordance with the TGO 54 under `clean` and `dirty` conditions, 2008-12

Please feel free to request these clinical data.

**TGA-registered or listed

Cleaning and disinfection of surfaces

as at: 8 July 2015

Field of application

MaxiWipes soaked with PlastiSept are especially suited for surfaces sensitive to alcohol of equipment such as treatment units (e. g. upholstery, head rests), suspension tables, in the X-ray area (e. g. chin rests at OPG) and for surfaces of acrylic glass (also for solarium beds).

Field of application

MaxiWipes soaked with MinutenSpray-classic are suited for surfaces compatible with alcohol if quick drying and a comprehensive disinfection are required.

Delivery forms

MaxiWipes

- REF 4614 250 MaxiWipes in box, dry
REF 4613 4 x 250 MaxiWipes refill, dry
(50 g/m², 17 x 26 cm)
for excellent absorption and
application of the active solution

MaxiWipes-L

- REF 4616 160 MaxiWipes-L in box, dry
REF 4615 4 x 160 MaxiWipes-L refill, dry
(50 g/m², 20 x 40 cm)
for excellent absorption and
application of the active solution
- REF 4617 Wall bracket for MaxiWipes/
MaxiWipes-L-Box



Distributor:

Biodegree Pty Ltd
Sydney, Australia